

Horizon[®] Imager Software

Release Notes

Version 1.8.5 Release Notes

Horizon version 1.8.5 software corrects two defects:

- ◆ When a DICOM Print SCU specified a multi-image sheet and did not provide an image for the first cell, versions 1.8.3 and earlier software failed to print the sheet.
- ◆ When a DICOM Print SCU specified a multi-image sheet with a magnification type of NONE, versions 1.8.3 and earlier software would omit scaling only for the first cell. All of the other images in the remaining cells would be incorrectly scaled.

Minor changes were made to the Horizon Engine firmware to support new edge-detection hardware. No changes were made affecting older edge-detection hardware.

In all other respects, version 1.8.5 is identical to version 1.8.3. See Version 1.8.3 Horizon Imager Software Release Notes (Codonics Part Number 901-084-008) for full details.

This software is compatible with all Horizon imagers including LifeImager[™] products. Any Horizon imager running software versions 1.5.x through 1.8.3 can be upgraded to version 1.8.5. Imagers running version 1.4.x software must first be upgraded to version 1.6.0 before loading version 1.8.5. If your Horizon imager is running version 1.4.x software, please contact Codonics Technical Support.

Get it all with just one call
1-800-444-1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice. Patents pending.

Copyright © 2006 by Codonics, Inc. Printed in the U.S.A. Part No. 901-084-009 Rev.A.



17991 Englewood Drive
Middleburg Heights, OH 44130 USA
(440) 243-1198
(440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023 JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295