

Codonics Horizon Operating Software

Technical Brief

Version 3.1.0 Release Notes

Summary

Horizon® 3.1.0 software improvements reduce the likelihood of SmartCard corruption and provides factory support for media keys 32 to 255.

Products Affected

Imagers with a serial number prefix of 53C, 54C, or 55C are affected. Please contact your Codonics representative for further information.

The Imager Software Version can be determined using the Control Panel by viewing the Imager Info screen from the Utilities menu.

NOTE: Horizon Imagers with serial number prefix 53C, 54C, or 55C cannot be downgraded to any previous software version below 3.0.0. Imagers with a serial number prefix 53C, 54C, or 55C cannot have their software upgraded to 4.0.0 or higher.

New Features (Adaptive Updates)

- ◆ None in this release.

Product Improvements (Perfective Updates)

- ◆ **Improved SmartCard Reliability.** Horizon reads the SmartCard at system startup and writes it back to the card at various times. The time needed to write to the card has been reduced. This will reduce the window for removing a SmartCard during a write operation, thus reducing the likelihood of SmartCard corruption.

- ◆ Horizon 3.1.0 software provides factory support for media keys 32 to 255.
- ◆ The FDA UDI GTIN (14-digit number) has been removed from Horizon software. Since Horizon software only works on Horizon imagers a UDI number is not required for software.

Defects Corrected (Corrective Updates)

- ◆ None in this release.

Known Common Issues

- ◆ Previous issues can be found in previous release notes.

Known Uncommon Issues

- ◆ Previous issues can be found in previous release notes.

Technical Support

If problems occur during software installation, please contact Codonics Technical Support at any time.

Phone: +1.440.243.1198

Email: support@codonics.com

Website: www.codonics.com

Get it all with just one call
800.444.1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.
Patents: www.codonics.com/ip/patents
Copyright © 2020 Codonics, Inc. Printed in the U.S.A.
Part No. 901-254-004.01.



17991 Englewood Drive
Middleburg Heights, OH
44130 USA
+1.440.243.1198
+1.440.243.1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Trading Co, Ltd.
317 XianXia Rd. Building B
Unit 1412
Changning Dist., Shanghai
P.R. China, 200051
86-21-62787701
86-21-62787719 Fax

Codonics Limited KK
AQUACITY 9F,
4-16-23, Shibaura
Minato-ku, Tokyo,
108-0023 JAPAN
81-3-5730-2297
81-3-5730-2295 Fax