

# Codonics Integrity Operating Software

## Release Notes

### Version 1.3.0 Release Notes

#### Summary

Version 1.3.0 software is the initial release of the Codonics® Integrity™ Medical Image Importer. The software is compatible with all Integrity systems.

#### Known Common Issues

This section details common issues with Integrity version 1.3.0 software that are likely to be experienced by most users.

- ◆ **There isn't a way to stop or to delete a study that is storing.** A user must wait until the store is complete before deleting the study.

#### Known Uncommon Issues

This section details uncommon issues with Integrity version 1.3.0 software that are unlikely to be experienced by most users.

- ◆ **After successfully changing the IP address from the user interface, a remote browser waiting icon shows indefinitely.** When a user changes the IP address from the user interface, they will need to browse to the new IP address and log back into Integrity.
- ◆ **Study deletion stops working if a study is reconciling or storing and the study needs to be deleted.** The situation will only occur with a full database of studies, and at the same time a study needs to be deleted that is locked, reconciling, queued, or storing.
- ◆ **A study can be locked if a user interrupts the Reconcile page from loading.** If a user selects a study, clicks the Reconcile button, and then interrupts the Reconcile page from loading by clicking on the Help button, the Reconcile page does not load but the Help screen does. However, when the user returns to the Studies page, the status of the study is locked for Editing instead of Ready.

- ◆ **A disc inserted upside-down results in no action on Integrity.** When a disc is inserted upside-down, the Integrity shows no recognition that a disc was inserted. The on-screen LEDs show no status change, and no status message is displayed.
- ◆ **Selecting a study that is in the process of storing causes user interface issues and the count to be off-by-one.** If a study is being stored and a user selects the same study while it is being stored, the study will move to the Stored screen, but the Delete, Store, and Reconcile buttons will remain selectable although there is no longer a selected study on the New screen. The lower left hand corner also reads 1 of 0 selected.
- ◆ **User Interface does not load properly for certain locales.** Version 1.3.0 supports English (en\_US) and Japanese (ja\_JP) user interfaces. Other locales are not fully supported and have not been fully verified.
- ◆ **A destination profile name that is longer than 16 characters will cause storing to be inactive.** When naming a destination profile, make sure that the name of destination in *destination.name of destination.txt* is less than 16 characters.
- ◆ **A user password must be longer than three characters.** When an admin sets a user password either from the user interface or on the SmartDrive, a password must be longer than three characters in order to work.

#### Technical Support

If problems occur during software installation, contact Codonics Technical Support between the hours of 8:30AM and 5:30PM EST (weekends and U.S. holidays excluded).

Phone: 440-243-1198

Email: support@codonics.com

Website: www.codonics.com

*Get it all with just one call*  
**1-800-444-1198**

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice. Patents pending.

Copyright © 2007 by Codonics, Inc. Printed in the U.S.A. Part No. 901-179-003 Rev. 01.



17991 Englewood Drive  
Middleburg Heights, OH 44130 USA  
(440) 243-1198  
(440) 243-1334 Fax  
Email info@codonics.com  
www.codonics.com

Codonics Limited KK  
New Shibaura Bldg. F1  
1-3-11, Shibaura  
Minato-ku, Tokyo, 105-0023 JAPAN  
Phone: 81-3-5730-2297  
Fax: 81-3-5730-2295