

Codonics Integrity Software Updates

Overview

This Technical Brief provides instructions for obtaining and installing software updates for the Codonics Integrity™ Medical Image Importer.

Obtaining the Installer Update File

1. Obtain the Update that you want to install from the Codonics web site or Codonics Technical Service.
2. Copy the Update installer file to a PC's hard drive. The file will be named with the Codonics Part Number, Name and Version.


Example: 025-078-001.01_lbl_3603_1.0.exe

3. Access the SmartDrive of the Integrity to update by connecting the Integrity's SmartDrive to the PC.
4. If an \updates folder does not exist on the SmartDrive, create one.
5. Transfer the downloaded file to the \updates folder on the SmartDrive.
6. Remove the SmartDrive and re-connect it to Integrity.


Installation Process

1. To assure that Integrity is not processing jobs during the installation, temporarily disconnect Integrity from network connections.
2. Attach a keyboard, mouse and monitor to the system.
3. Minimize the User Interface window by pressing the **Windows**® key and **M** key at the same time.
4. Press the **Windows** key and **E** key at the same time - this will bring up a My Computer window.
5. Browse to the SmartDrive, which is listed in the right pane as a Removable Drive with the name CODONICS. Left-click the SmartDrive and press **Enter**.



6. Identify the Installer File that was downloaded to the SmartDrive, left-click the Installer File and press **Enter**.

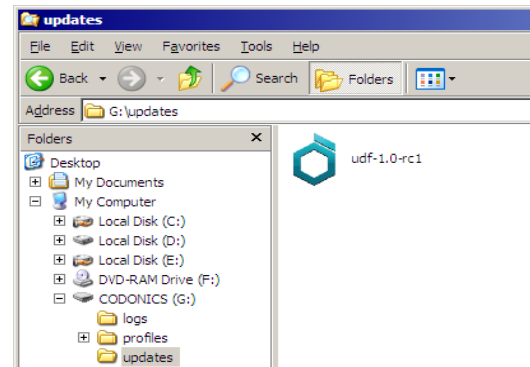


7. You will be asked if you want to continue with the update. If you select **OK**, the Codonics Update Installer will install the necessary files. If it is necessary for the update, the Installer will stop Integrity software.

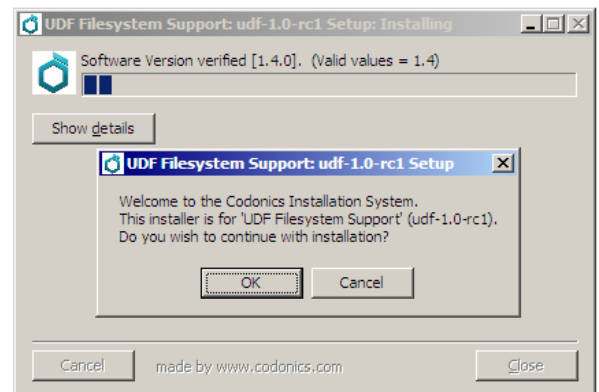

NOTE: If the Installer File was not downloaded correctly, or if this is a system upon which the Update cannot be installed, a dialog box will display an error message. Clicking **OK** on the dialog box will exit the Installer. Examples:

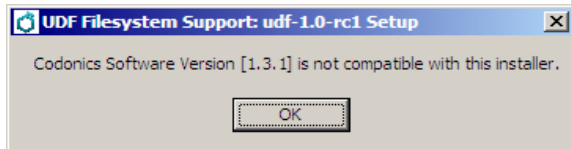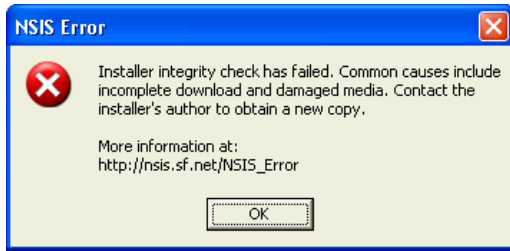

- When the installation completes correctly, a dialog box will appear asking if you want to reboot. If you have multiple updates to install, click **No** and repeat steps 5 through 7 for each update. If this is the only update or you are finished, click **Yes** and the system will reboot.

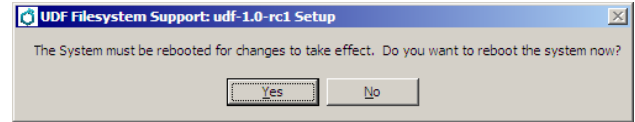

- If necessary, remove keyboard, mouse and/or monitor. Reconnect the network cable.

Technical Support

If problems occur that are not covered by this Technical Brief, contact Codonics Technical Support between the hours of 8:30AM and 5:30PM EST (weekends and U.S. holidays excluded).

Phone: 440-243-1198
Email: support@codonics.com
Website: www.codonics.com

Get it all with just one call
1-800-444-1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice. Patents pending.

Copyright © 2010 by Codonics, Inc. Printed in the U.S.A. Part No. 901-215-002.01



17991 Englewood Drive
Middleburg Heights, OH 44130 USA
(440) 243-1198
(440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023 JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295