

# Calibration Profile Update Diskette

---

**Technical Brief**

## **Calibration Profile Update Diskette**

Each printer in the NP-1660 family is tuned at the factory for optimal image quality. Part of this process is to characterize the thermal print head and create a "calibration profile." The calibration profile is used by the printer to compensate for small differences within the print head.

The Calibration Profile Update Diskette included with your NP-1660 printer contains a copy of the calibration profile which is also resident on the printer's hard disk. This diskette will be needed if the printer software is reloaded, or if the printer receives a complete software upgrade.

The calibration profile on the diskette is unique to the printer with which it is shipped. **It should not be used with other Codonics printers.**

The calibration profile can be loaded onto the printer from the diskette using the **Update Diskette Installation Procedure**.

## **Important Calibration Profile Update Diskette Notes**

- ◆ There is no need to load the calibration profile unless the printer software is completely reloaded or upgraded (an upgrade normally consists of ten or more diskettes).
- ◆ There is no need to load the calibration profile if the printer receives a software update from an Update Diskette (an update normally consists of a single diskette).
- ◆ The diskette should be stored in a safe, well known place.
- ◆ **Remember, the Calibration Profile Update Diskette should only be used with the printer with which it was shipped.**

---

*Get it all with just one call*  
**1-800-444-1198**



**CODONICS**

*We bring the future into focus*

17991 Englewood Drive  
Middleburg Heights, OH 44130 USA  
440/243-1198  
440/243-1334 Fax  
Email [info@codonics.com](mailto:info@codonics.com)