

Overview

The Codonics NP-1600 provides a utility that aids in restoring the unit to factory default settings. This helps users who are unsure how they may have changed the settings, System Administrators who must manage a unit that has been modified by multiple users, and sales personnel who want to make sure that a demonstration unit is in the proper state before entering a customer site. The Level-0 Reset feature is also useful for System Administrators or sales personnel who have forgotten the root password on units where one was installed.

Level-0 Reset

Level-0 Reset restores many printer settings to the factory defaults.

Primary Functions:

- ◆ Clears all print queues.
- ◆ Resets all network parameters to factory default values.
- ◆ Restores the IP address to factory default (192.0.2.0) in the host file.
- ◆ Clears the system directory:
 - ◆ Restores root (printer default) image processing parameters.
 - ◆ Removes user-defined image processing parameters.
- ◆ Resets parallel port parameters to factory defaults.
- ◆ Deletes all files from data partition of hard drive: **/data/tmp** and **/data/parallel** directories.
- ◆ Removes all passwords, including any root password, and resets **passwd** file to factory default.
- ◆ Clears log files.
- ◆ Removes all **core** files on the hard drive.

Note: Level-0 Reset does **not** affect any Activation Keys. Refer to the [Activation Keys Technical Brief](#) for information on removing Activation Keys.

Front Panel Sequence

Use the following front panel key sequences to perform a Level-0 Reset of the printer. When an **<Up-arrow>** or **<Down-arrow>** is indicated, press the key repeatedly until the correct response is showing on the LCD. Start from the READY display.

<u>Use Key</u>	<u>Until LCD Displays</u>
<Setup>	SETUP: NETWORK
<Up-arrow>	SETUP: KEYS
<Setup>	KEY:
Enter Key Code: R 0 _ _ _ _ _ _ _ _	Where the _ characters represent blank spaces.
<Up-arrow> or <Down-arrow>	To change 1st. letter of activation key.
<Setup>	Move to next letter of activation key.
	Repeat the use of arrow and setup keys until all letters are entered. After the last letter of the key is entered, the LCD will display:
	SETUP: KEYS
<Up-arrow>	SETUP: EXIT
<Setup>	READY

After the Level-0 Reset Key has been installed and the printer's front panel displays READY, cycle power **off** for 5 seconds then **on** to reboot.

Get it all with just one call
1-800-444-1198

