

Codonics Virtua Print Carriage

**Product
Bulletin**

No. 206

Overview

Codonics Virtua® Medical Disc Publishers will automatically move the print carriage to its home position when the cover is lifted. After the cover has been lifted for five minutes, the print carriage will automatically move to the left side of the recorder. This function prevents the ink from drying prematurely.

CAUTION: The print carriage will automatically move to its home position on the left side of the recorder. Please be aware of the expected movement.

The behavior described in this Product Bulletin is only for Virtua systems that utilize a separate print head and individual ink cartridges for each color. These systems can be identified by the first three characters in the serial number of the Recorder, 83C. See Virtua Reference Guide, part number 900-567-001, for more details on newer Virtua systems.

To prevent the print carriage from returning to its home position after opening the cover, immediately press the Ink Cartridge Load button. The print carriage will remain still for several minutes after the Ink Cartridge Load button is pressed.

NOTE: If ink cartridges need to be replaced after the print carriage returns to its home position, please refer to the Replace an Ink Cartridge section of the Virtua Reference Guide, part number 900-567-001. The Ink Cartridge Load button will re-position the print carriage for easy access for ink cartridge replacement.

Technical Support

Please contact Codonics Technical Support at any time.

Phone: +1.440.243.1198

Email: support@codonics.com

Website: www.codonics.com

Get it all with just one call
1.800.444.1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.

Copyright © 2012-2013 Codonics, Inc. Printed in the U.S.A. Part No. 903-302-002.02.



We bring the future into focus

17991 Englewood Drive
Cleveland, OH 44130 USA
+1.440.243.1198
+1.440.243.1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023 JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295