# **CODONICS**

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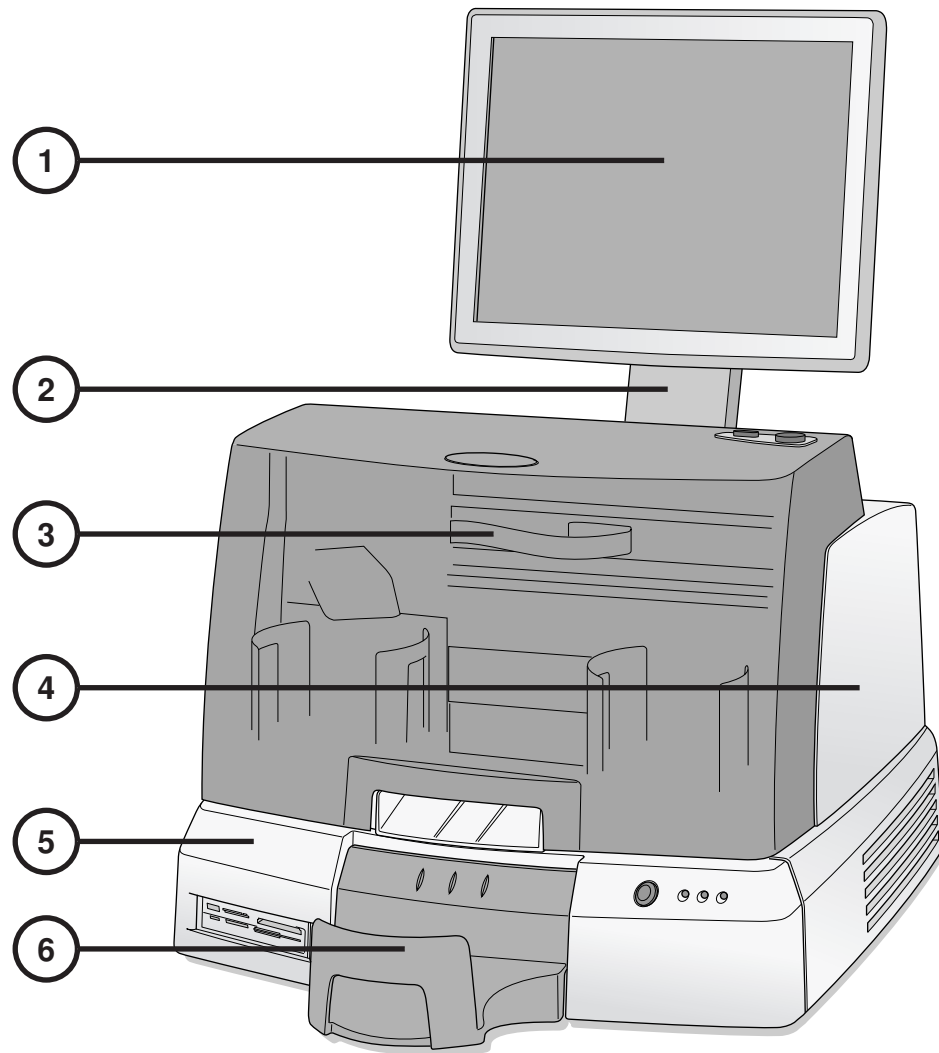
## **Table of Contents**

Composants .....	1
Assemblage.....	6
Démarrage .....	14
Arrêt .....	16
Chargement Media .....	17
Configuration .....	20
Configuration DICOM Host .....	31
Installation Logiciel .....	32
Dépannage .....	34

# Composants

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```
[ SiteProfile ]
address0 = Codonics Imaging Center
address1 = 17991 Englewood Drive
address2 = Suite D
address3 = Middleburg Heights, OH 44130
customLabelFileLocation =
defaultImagerProfile =
defaultJobProfile =
demoSiteBurnDirectory = c:\data\cc\spd
inactivityTimeout = 60
isDemoSite = false
logoFileLocation =
siteDisclaimer = This disc contains confidential medical information. If you
are not the intended recipient or an authorized user, please contact the site
shown on this label.
siteName = Codonics
phoneNumber = 1-800-444-1198
url = www.codonics.com
```

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## Graver des Examens

The screenshot shows the CODONICS software interface. At the top, there is a header bar with the CODONICS logo, a timer at 0:00, and a 'Ready' status indicator. Below the header, there are three tabs: 'Studies', 'Utilities', and 'Help'. The 'Studies' tab is active. Underneath, there are three sub-tabs: 'New', 'Today', and 'Yesterday'. The 'Today' sub-tab is selected. A table displays a list of exams with columns for Patient, Physician, Modality, Exam, Received, Status, and Actions. Two exams are listed: BARRY, JOHN and BORG, MARY. The BORG, MARY exam is selected, indicated by a green checkmark in the first column. Below the table, there are buttons for 'Select All', 'Unselect All', and '1 of 2 selected'. At the bottom, there are buttons for 'Delete...' and 'Record'. Five numbered callouts (1-5) point to specific elements: 1 points to the 'Studies' tab, 2 points to the 'Studies' sub-tab, 3 points to the 'Today' sub-tab, 4 points to the 'BORG, MARY' row in the table, and 5 points to the 'Record' button.

Patient	Physician	Modality	Exam	Received	Status	Actions
BARRY, JOHN	FELANI	CR (1)	P543215 11/07/06	3:58 PM	Ready	[Trash] [Search]
<input checked="" type="checkbox"/> BORG, MARY	PATEK	MR (64)	P543213 11/07/06	3:44 PM	Ready	[Trash] [Search]

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The screenshot shows the Codonics Virtua software interface. At the top left is the Codonics logo. The top right shows system information: 'virtua 14/12/06 11:14 AM' and a 'Log Out' button. Below this are 'Studies', 'Utilities', and 'Help' tabs. The main area is divided into two sections: 'Selections' and 'Options'.

**Selections Table:**

Patient	Physician	Modality	Exam	Received	Status
BORG, MARY	PATEK	MR (64)	P543213 11/07/06	3:44 PM	Ready

Below the table, it indicates '1 Study' and 'Page 1 of 1' with navigation arrows.

**Options Section:**

Options: 2 physician, 1 patient, 0 anonymized, 0 legal

<input checked="" type="checkbox"/> Job Profile: None Selected	<input checked="" type="checkbox"/> Viewer: Codonics Viewer	<input checked="" type="checkbox"/> Media Type: CD
<input checked="" type="checkbox"/> Disc Label: Ink Saver	<input checked="" type="checkbox"/> Make Auto-Run	<input checked="" type="checkbox"/> Verify Discs

At the bottom of the 'Options' section are 'OK' and 'Cancel' buttons. A circled number '12' points to the 'OK' button.

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© Copyright 2006-2009 par Codonics, Inc. imprimé au USA