# CODONICS

**Virtua® Başvuru Klavuzu**

## ***İçindekiler***

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```
[ SiteProfile ]
siteProfileName = default
address0 = Codonics Imaging Center
address1 = 17991 Englewood Drive
address2 = Suite D
address3 = Middleburg Heights, OH 44130
customLabelFileLocation =
defaultImagerProfile =
defaultJobProfile =
demoSiteBurnDirectory = c:\data\cc\spd
inactivityTimeout = 60
isDemoSite = false
logoFileLocation =
siteDisclaimer = This disc contains confidential medical information. If you
are not the intended recipient or an authorized user, please contact the site
shown on this label.
siteName = Codonics
phoneNumber = 1-800-444-1198
url = www.codonics.com
```

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Studies Discs Query/Retrieve

New Today Yesterday All

Patient	Physician	Modality	Exam	Received	Status	Actions
BARRY, JOHN	FELANI	CR (1)	P543215 11/07/06	3:58 PM	Ready	
<input checked="" type="checkbox"/> BORG, MARY	PATEK	MR (64)	P543213 11/07/06	3:44 PM	Ready	

Select All Unselect All 1 of 2 selected Page 1 of 1

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
The screenshot shows the Codonics Virtua software interface. At the top left is the Codonics logo. The top right shows a timer at 0:00, a 'Ready' status, and a 'Log Out' button. Below the timer are 'CD' and 'DVD' indicators. The main area is divided into 'Studies', 'Utilities', and 'Help' tabs. The 'Studies' tab is active, showing a table of study selections. Below the table is a '1 Study' indicator and navigation buttons. The 'Options' dialog box is open, showing a table of options with checkboxes. A circled number '6' points to the 'Edit Options...' button in the top right of the dialog box.

Patient	Physician	Modality	Exam	Received	Status
BORG, MARY	PATEK	MR (64)	P543213 11/07/06	3:44 PM	Ready

Copies:	1 physician	0 patient	0 anonymized	0 legal
<input checked="" type="checkbox"/> Job Profile: default	<input checked="" type="checkbox"/> Viewer: Codonics Viewer	<input checked="" type="checkbox"/> Media Type: CD	<input checked="" type="checkbox"/> Verity Discs	
<input checked="" type="checkbox"/> Disc Label: Physician	<input checked="" type="checkbox"/> Make Auto-Run			

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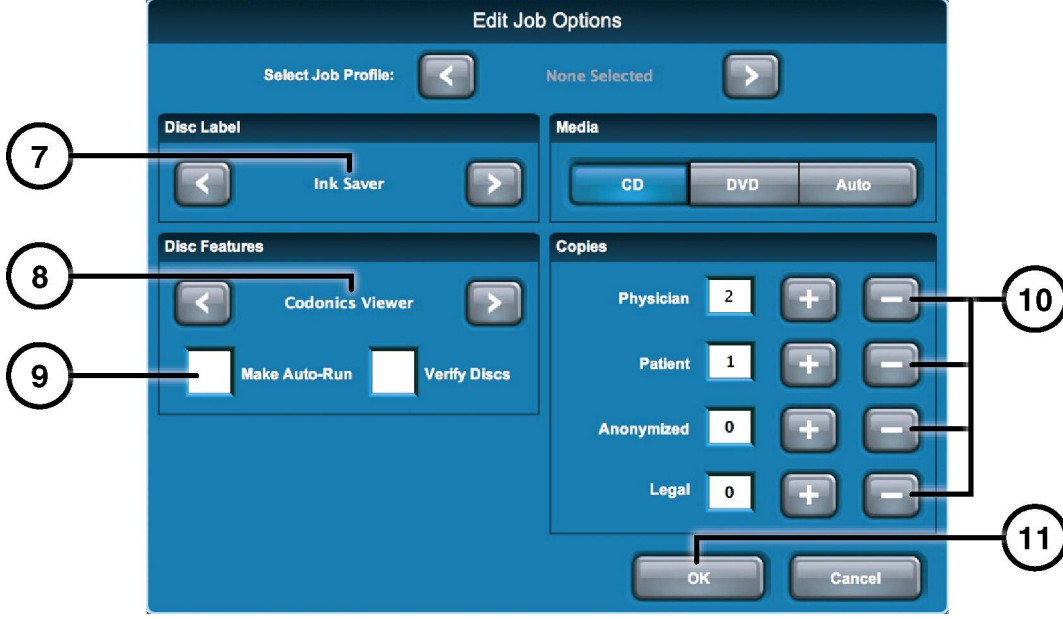Studies Utilities Help

### Selections

Patient	Physician	Modality	Exam	Received	Status
BORG, MARY	PATEK	MR (64)	P543213 11/07/06	3:44 PM	Ready

1 Study Page 1 of 1

### Options

Edit Options...

Copies:	2 physician	1 patient	0 anonymized	0 legal	
<input checked="" type="checkbox"/> Job Profile: None Selected	<input checked="" type="checkbox"/> Viewer: Codonics Viewer	<input checked="" type="checkbox"/> Media Type: CD	<input checked="" type="checkbox"/> Disc Label: Ink Saver	<input checked="" type="checkbox"/> Make Auto-Run	<input checked="" type="checkbox"/> Verify Discs

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Tüm kayıtlı ve kayıtsız ticari markalar ilgili sahiplerinin mülkiyetindedir. Özellikler haber verilmeden değiştirilebilir. Patent Copyright © 2007 – 2010 by Codonics, Inc. Printed in U.S.A. Part Number 900-367-014.01.